


# *New Clinical App Update*

# New Clinical App Features

## Index

1.	Guide - Patient Identification		3
2.	New Heel Cut Out	7	
3.	New Modifications Available		8

# Guide - Patient Identification

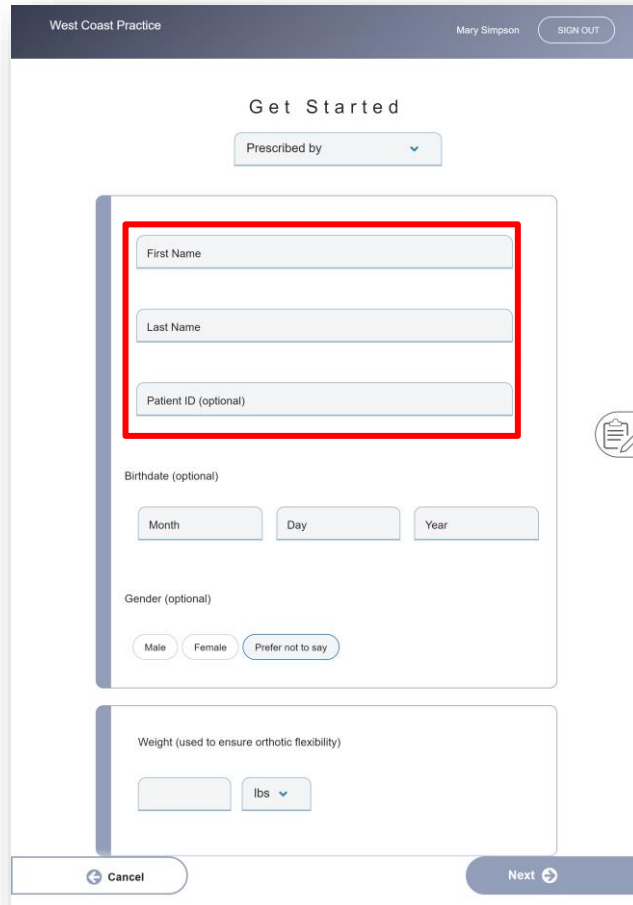

West Coast Practice Mary Simpson SIGN OUT

Get Started

Prescribed by ▼

First Name

Last Name

Patient ID (optional)

Birthdate (optional)

Month Day Year



Gender (optional)

Male Female Prefer not to say

Weight (used to ensure orthotic flexibility)

lbs ▼

Cancel Next

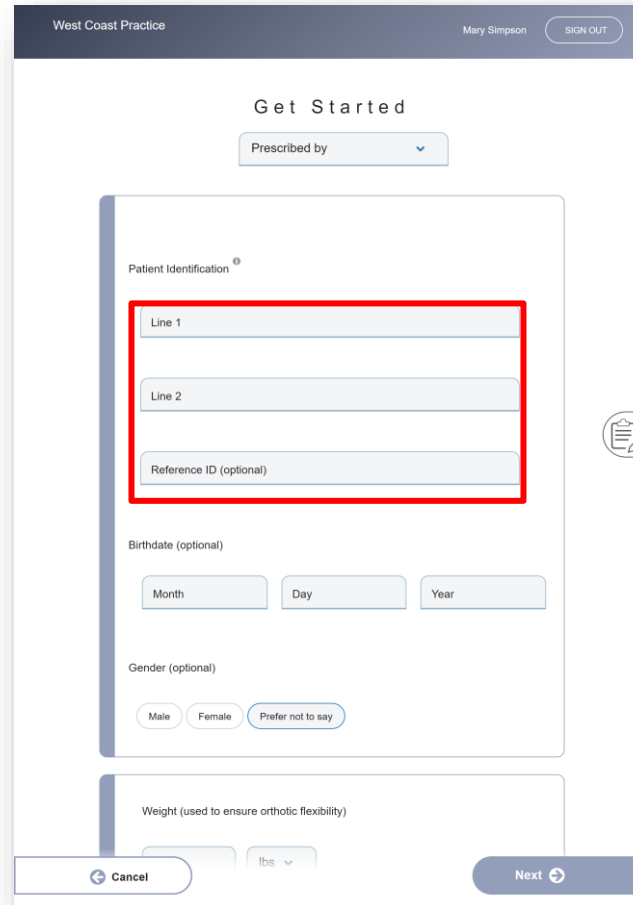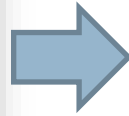

West Coast Practice Mary Simpson SIGN OUT

Get Started

Prescribed by ▼

Patient Identification ®

Line 1

Line 2

Reference ID (optional)

Birthdate (optional)

Month Day Year

Gender (optional)

Male Female Prefer not to say

Weight (used to ensure orthotic flexibility)

lbs ▼

Cancel Next

“First Name” and “Last Name” fields have been changed to say, “Line 1” and “Line 2”. The optional Patient ID field has been changed to “Reference ID (optional)”

You can continue to enter first and last name or use unique identifiers in those lines.

Please use these guidelines for deciding how to use these new fields:

- 1 Line 1 and Line 2 are mandatory
- 2 The fields are limited to 25 characters
- 3 Both lines will be printed on the orthotic
- 4 All three fields are searchable on your patient page.

Please see the examples of how you can use these new field options.

# Example 1

## Entering First and Last Name only

West Coast Practice

Mary Simpson

SIGN OUT

Get Started

Prescribed by

1 John

2 Smith

3 ABCD9876

Birthdate (optional)

Month

Day

Year

Gender (optional)

Male

Female

Prefer not to say

Cancel

Next

West Coast Practice

Mary Simpson

SIGN OUT

All Clinicians

Privacy

Patients

All

Ready


Scanned

Saved Rx

Orders

Patient Info		DOB	Reference ID	Last Updated
1	2		3	
John Smith			ABCD9876	6/19/2020 10:03 am
Mike Kelly		4.6.1987	JILKL-MM432	6/18/2020 11:12 am
Joe Knoll		5.10.1975	UJHYT-MM865	6/20/2020 10:03 am

How it appears on Patients page in app

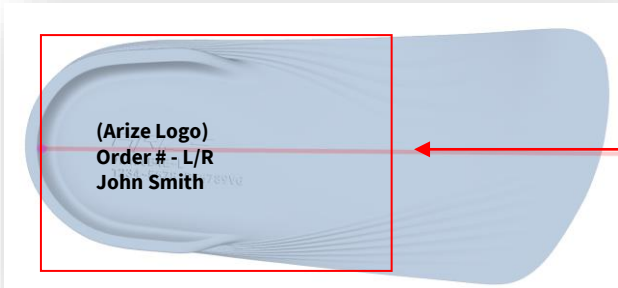

How it appears on the shell

# Example 2

Entering unique identification numbers and a PO

West Coast PracticeMary SimpsonSIGN OUT

Get Started

Prescribed by

1 1234-5678

2 PO B56789VG

3 ABCD9876

Birthdate (optional)

Month

Day

Year

Gender (optional)

Male

Female

Prefer not to say

Cancel

Next

West Coast PracticeMary SimpsonSIGN OUT

All Clinicians

Privacy

Patients

All

Ready


Scanned

Saved Rx

Orders

Patient Info	DOB	Reference ID	Last Updated
<div>1 1234-5678</div> <div>2 PO B56789VG</div>		<div>3 ABCD9876</div>	6/19/2020 10:03 am
Mike Kelly	4 6.1987	JILKL-MM432	6/18/2020 11:12 am
Joe Knoll	5.10.1975	UJHYT-MM865	6/20/2020 10:03 am

How it appears on Patients page in app

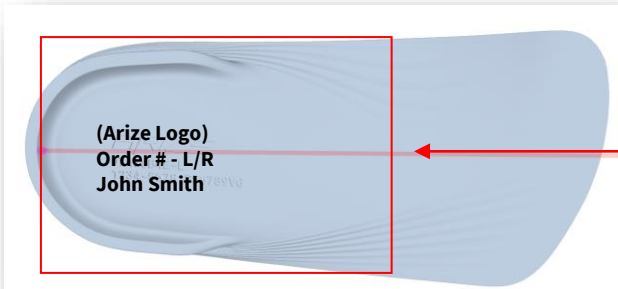

How it appears on the shell

# Example 3

## Entering First and Last name and Clinic Information

West Coast Practice

Mary Simpson

SIGN OUT

Get Started

Prescribed by

1

John Smith

2

Your Podiatry Clinic

3

ABCD9876

Birthdate (optional)

Month

Day

Year

Gender (optional)

Male

Female

Prefer not to say

Cancel

Next

West Coast Practice

Mary Simpson

SIGN OUT

All Clinicians

Privacy

Patients

All

Ready


Scanned

Saved Rx

Orders

Patient Info	DOB	Reference ID	Last Updated
<div>1</div> <div>John Smith</div> <div>2</div> <div>Your Podiatry Clinic</div>		<div>3</div> <div>ABCD9876</div>	6/19/2020 10:03 am
Mike Kelly	4.6.1987	JILKL-MM432	6/18/2020 11:12 am
Joe Knoll	5.10.1975	UJHYT-MM865	6/20/2020 10:03 am

How it appears on Patients page in app

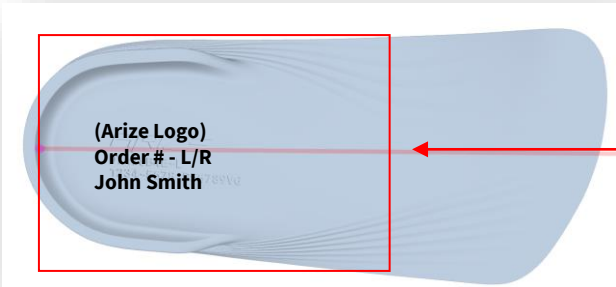

How it appears on the shell

# Heel Cut Out

## New Modification

You now have the ability to order a heel cutout in the UI. The cutout is 1" in diameter and automatically applies when selected. You will be able to view the cutout in the Visualizer tool before hitting submit.

### SHELL MODIFICATIONS

#### Shell Rigidity ⓘ

Semi-Rigid

#### Shell Length ⓘ

Met Length

Sulcus Length

Full Length

#### Heel Cut Out ⓘ

N/A

L

R

B/L

#### 1st Ray Cut Out ⓘ

N/A

L

R

B/L

#### Flexible Morton's Ext ⓘ

N/A

L

R

B/L

#### Semi-Rigid Morton's Ext ⓘ

N/A

L

R

B/L

#### Reverse Morton's Ext ⓘ

N/A

L

R

B/L

#### Heel Seat ⓘ

1/8"

1/4"

3/8"

1/2"

5/8"

3/4"

#### Arch Modification ⓘ

N/A

Increase 1/8"

Decrease 1/8"

#### Medial Flange ⓘ

N/A

L

R

B/L

#### Lateral Flange ⓘ

N/A

L

R

B/L



# New Modifications Available

Type of finishing instructions received frequently Save in Preferred Prescriptions Manually applied, will not show up in visualizer  
Three section: Shell, cover and Accommodations

SHELL MODIFICATIONS

Shell Rigidity ⓘ  
Semi-Rigid

Shell Length ⓘ  
Met Length  
Sulcus Length  
Full Length

Heel Cut Out ⓘ  
N/A  
L  
R  
B/L

1st Ray Cut Out ⓘ  
N/A  
L  
R  
B/L

Flexible Morton's Ext ⓘ  
N/A  
L  
R  
B/L

Semi-Rigid Morton's Ext ⓘ  
N/A  
L  
R  
B/L

Reverse Morton's Ext ⓘ  
N/A  
L  
R  
B/L

Heel Seat ⓘ  
1/8"  
1/4"  
3/8"  
1/2"  
5/8"  
3/4"

Arch Modification ⓘ  
N/A  
Increase 1/8"  
Decrease 1/8"

Medial Flange ⓘ  
N/A  
L  
R  
B/L

Lateral Flange ⓘ  
N/A  
L  
R  
B/L

Other Shell Modifications\* ⓘ  
⌵

Other Shell Modifications\* ⓘ  
+ Widen forefoot at shell  
+ Cutout at 5th MPJ  
+ Cut out at base of 5th met  
+ Accommodate the base of 5th met  
+ Accommodate the navicular  
+ Accommodate the heel

Click on the down arrow to view the options



# Other Shell Modifications

Modification	Description
Widen forefoot at shell	
Cutout at 5th MPJ	This will create a rounded cutout in the shell at the distal 5 <sup>th</sup> mpj area
Cut out at base of 5th met	This is the same as a 5 <sup>th</sup> metabase accommodation
Accommodate the base of 5th met	This will create a well or button-out area under the 5 <sup>th</sup> metabase
Accommodate the navicular	This will create a well or button-out area under the navicular
Accommodate the heel	Should only be used with a RF heel post, this modification creates a depression under the heel to offload

# Other Coverings

Modification	Description
Widen / Lengthen top cover throughout 2 sizes	For example, a size 12 cover template will be used on a size 10 shell
Widen top cover at forefoot only 2 sizes	We will leave the forefoot portion of the top cover wider / longer at the forefoot area only
Add navicular wing	This is a curved raised area of the top cover along the medial arch of the orthotic in the area of the navicular. It will pad navicular prominences that lie above the shell.

# Other Accommodations

Modification	Description
1/16" poron backfill	Poron fill for selected accommodations. Not needed usually if a midlayer of 1/16" or greater is selected.
EVA reinforcement from heel to mets	This could affect turnaround times. Similar results can be achieved by increasing rigidity of the shell as a consideration.
EVA arch reinforcement	This could affect turnaround times. Similar results can be achieved by increasing rigidity of the shell as a consideration.
1/16" EVA distal FF extension	
Extrinsic tip post lateral	Used when any extrinsic or intrinsic FF posting is selected to balance the FF to the RF.
Extrinsic tip post medial	Used when any extrinsic or intrinsic FF posting is selected to balance the FF to the RF.
Extrinsic FF post lateral	Used when any extrinsic or intrinsic FF posting is selected to balance the FF to the RF.
Extrinsic FF post medial	Used when any extrinsic or intrinsic FF posting is selected to balance the FF to the RF.